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Device for ultrasonic aerosol therapy with a very refined design and new technology, characterized by extreme noiselessness. With the traditional ultrasonic aerosol devices, compounds for inhalation are transformed, by means of oscillation of a piezoelectric crystal embedded in the device, into liquid micro-drops dispersed in air. In this aerosol therapy device, this transformation occurs by means of the flow of the drug in the micro-holes of a foil. The microdroplets of the drug, ranging in size between 1 and 5 µm, are able to penetrate deep into the upper and lower respiratory tract, targeting the active ingredients where needed.

Aerosol therapy is effective for the treatment of lung disorders, tracheitis, asthma, bronchitis, sinusitis and allergies. It can also instantly relieve symptoms caused by colds and allergic irritations, decreasing the urge to cough, and alleviating sinus pain and pressure, facilitating expectoration. It is an excellent device for home use, is practical and easy to use. It is built in compliance with current European standards regarding general requirements for the safety of medical electrical equipment (Standard EN 60601-1).

The product's small size makes it extremely convenient for those who must use it several times a day and for those who travel. The use of oil-based drugs or those with solid elements as active ingredients (suspensions), viscous solutions or essential oils, is not recommended. **Always follow the doctor's instructions for the type of medicine to use, the dosage, frequency and duration of inhalations.**

BEFORE USE IT IS IMPORTANT TO READ THE INSTRUCTIONS AND WARNINGS CONTAINED IN THIS BOOKLET CAREFULLY AND KEEP IT IN A SAFE PLACE FOR FUTURE REFERENCE.

KEY TO SYMBOLS

CE 0425

In compliance with European legislation on medical devices

Manufacturer

Caution! Read the user instructions carefully

Symbol for "Type BF applied parts"

Class II appliance

Batch production number

Not for use outdoors

IP22: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

GENERAL WARNINGS

Using any electrical appliance entails compliance with a number of fundamental rules:

- Unplug the appliance from the mains straight after use and before commencing cleaning or inserting/removing accessories.
- Do not unplug by tugging on either power cord or appliance.
- If an electrical appliance falls into the water, do not attempt to retrieve it; instead, unplug immediately.
- Do not use the appliance while bathing or showering.
- Never immerse the appliance in water or other liquids.
- **Never leave the machine running unattended. Do not allow the appliance to be used by children, invalids or disabled individuals without adult supervision.**
- This appliance must be used solely for the purpose for which it was designed and in accordance with the operating instructions. Any other use shall be considered improper and hence hazardous. The manufacturer cannot be held responsible for any damage resulting from improper or incorrect use.
- Keep all parts of the packaging away from children (choking hazard).
- Before using the appliance, make sure it is intact and does not feature any visible signs of damage. If in any doubt, do not use the appliance and contact your dealer.
- In the event of a fault and/or irregular operation, turn the appliance off at once without tampering with it. Always contact your dealer for repairs.
- Keep the appliance away from sources of heat.
- Make sure your hands are dry when unplugging or plugging in the appliance and when turning the function control knob.
- This device shall be used only with the voltage stated in this booklet.
- Device with enclosure not protected against the penetration of liquids.
- Do not use the device close to high-frequency electromagnetic emitters.
- Do not use the appliance in the presence of flammable gases and in very dusty areas.
- **Do not use the device, if in the nebulization chamber there is not liquid or if the liquid is present in limited quantities.**
- Do not use powder medicinal products, powder medicinal products associated to liquid ones, or acid medicinal products with this appliance. Do not atomise water. Always use saline.
- **Always clean the appliance after each use in order not to let the residual medicinal products solidify and damage the mechanism.**
- During inhalation, keep the unit away from the eyes because some medicines can irritate and / or burn eyes.
- The device must only be used with the original accessories indicated in the instruction manual.
- Power outages, unexpected problems or other unfavourable conditions can make the device unusable. Therefore, we recommend keeping a spare device or substitute medicine (as agreed upon with the doctor).
- Do not use the device in a room where a spray was previously used. If you still wish to use the unit, we recommend airing the room before starting treatment.

IMPORTANT WARNINGS

- In case of diabetes or other illnesses, it is suggested to consult your physician before undergoing the inhalation therapy.
- Pregnant women should consult their physician before undergoing the inhalation therapy.
- Before using a preparation, clear with your physician, if it is suitable for the person to use it. Use only medicines prescribed or suggested by your physician.
- To dilute the preparation, use exclusively an isotonic solution of sodium chloride, since otherwise

respiratory problem may arise.

ACCESSORIES / COMPONENTS

- 1) "O/I" Button
- 2) Operating ("O/I") and low battery light
- 3) Atomisation chamber cover
- 4) Atomisation chamber
- 5) Atomisation dispenser
- 6) Atomisation chamber fastener
- 7) Mouthpiece
- 8) Adult mask
- 9) Child mask
- 10) Mask attachment
- 11) Battery compartment
- 12) Mains outlet
- 13) Transformer for mains use (not provided)
- 14) Convenient case
- 15) Batteries

TECHNICAL SPECIFICATIONS

Power supply

- Included: 2 x 1.5V AA LR6 alkaline batteries
- Battery autonomy: about 100 minutes of continuous use
- Mains operation (with transformer purchased separately): INPUT: 100-240V / 50-60 Hz - 0.15A / OUTPUT: DC 6.0V --- 0.4A

Operating conditions:

- Temperature: min 5°C; max 40°C
- Humidity: 15%-93% without condensation

Storage conditions:

- Temperature: min. -25°C; max. 70°C
- Humidity: < 93% without condensation
- Ultrasonic frequency: 100 kHz
- Atomisation: ≥ 0.25 ml/min.
- Percentage of aerosol in inhalation: 50% of particles < 5µm
- Average particle size: 4.8µm
- Maximum volume of inhalant: max 10ml = 10cc

INSTRUCTIONS FOR USE

This device can be used with 2 1.5V AA LR6 alkaline batteries or with the mains transformer (not provided but sold separately, Laica part number ANE033).

Using with batteries:

- 1) **Before each use, thoroughly clean and disinfect** the atomisation chamber, lid, connection, mouthpiece and masks according to the paragraph "Cleaning and Maintenance".
 - 2) Lift the cover of the battery compartment by gently pressing the tab on the lid and then pulling up. Insert the 2 1.5 V AA batteries, following the polarity indications. Close the cover. **Do not use rechargeable batteries.**
 - 3) Place the device on a flat surface.
 - 4) Place the atomisation chamber on the main unit and press it slightly downwards until a "click" is heard (fig. 1). Open the cover of the atomization chamber by pulling towards outside the small hook. The capacity of the nebulization chamber is 10 cc. Pour the drug paying attention not to exceed the maximum capacity, (the indications are born on the chamber itself), in order to prevent the liquid from flowing out and penetrating into the device itself damaging it (fig. 4). Furthermore, carefully follow the instructions for use and dosing of the product used. If this product has to be diluted with an isotonic solution of sodium chloride, pour first of all the solution in the nebulization chamber and then add the product dosing it as indicated.
- Caution! It is possible to use the device only if the drug has been poured into the nebulization chamber!**
- 5) Close the atomization chamber with the proper cover, paying attention to hook it correctly.
 - 6) At this point, insert the mouthpiece or the connection and adult/child mask on the lid as shown in figures 2 and 3.

For inhalation with the mouthpiece:

In case of disorders of the lower respiratory tract, i.e. cough or bronchial infections, use the mouthpiece.

- Sit upright.
- Insert the mouthpiece in the atomisation dispenser (see fig.2).
- Tighten the mouthpiece well with the lips.
- Inhale slowly and deeply through the mouth and exhale through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled. During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mouthpiece from the mouth and then reinsert it.

Caution! During inhalations using the mouthpiece, some medicinal product may deposit within the same.

In this case, it is suggested to proceed as follows after the treatment:

- 1) Disconnect the mouthpiece from the atomisation supplier paying attention not to let any medicinal product come out.
- 2) Re-position the medicinal product previously deposited in the mouthpiece within the atomisation chamber.
- 3) Perform the inhalation of the residual medicinal product, if greater than 0.5 cc.

In case of inhalation with mask:

The mask for inhalations is suitable for the treatment of the nasal and pharyngeal area.

- Sit upright.
- Insert the mask connection between the atomizer and the mask and then connect both parts.
- Keep the mask lightly pressed against the nose; the mask has to cover mouth and nose without squashing them.
- Inhale and exhale slowly and deeply through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled. During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mask for a while and then place it again on nose and mouth.

- 7) Switch on the device by pressing the "O/I" key. Make sure that the warning light is on (light blue color). During inhalation, it is possible to bend the device up to a max. angle of 45°C forwards, backwards or on the side without jeopardizing the nebulization.
- However, it is not recommended to shake a device with full nebulization chamber, in order to avoid damages due to any eventual penetration of liquids into the device. Immediately interrupt the treatment, if you feel uneasy during the procedure.
- 8) The appliance has an automatic stop device that disables the unit when the medicine is almost finished. In any case, turn the unit off when the atomisation flow has stopped. Do not try to turn the unit on if the medicine is empty since this could cause damage. In any event, the unit is equipped with a control device. You can turn the unit off at any time by pressing the "O/I" button.

Important:

- An inhalation treatment should not last more than 15 minutes. Stop the inhalation by pressing the "O/I" button even if the medicine is not completely empty. Allow the unit to cool for an hour to prolong the life of the mechanism that atomises the medicine, then continue the treatment.
 - Like most aerosol therapy appliances, at the end of the inhalation treatment, a certain amount of the drug (about 0.2 ml) will remain in the atomisation chamber. This is perfectly normal. This amount of the drug, also called the residual volume, cannot be atomised.
- 9) Allow the unit to cool for a few minutes and proceed with cleaning as indicated in the paragraph "Cleaning and maintenance" after each treatment.

Attention:

- **Do not try to turn the unit on if the batteries are dead (orange low battery light lit) or if there is no more medicine as this may cause damage. In any event, the unit is equipped with a control device.**
 - **If the batteries are low, the unit will turn off automatically after a short period of time, even if there is still some inhalation solution in the atomisation chamber.**
 - The battery charge decreases even if not used, so remove the batteries if not using the unit for long periods of time.
 - Remove the dead batteries by gently pressing on the tab on the battery compartment cover and then pulling it up. Remove the batteries and dispose of them as special waste at those locations indicated for recycling.
- For more information regarding the disposal of used batteries, contact the store where you purchased the appliance that contained the batteries or the City or local waste disposal service.

Attention: For greater safety, removal of the batteries by children under 12 years of age is NOT recommended.

The atomisation chamber is made of consumable material and thus its useful life is dependent upon several factors: use time, if used with more/less oily drugs, if maintenance and cleaning are performed properly and after each use. The atomisation chamber can be used 500 times or for a standard therapy use period of 1 year.

Use with mains transformer (not provided but sold separately. Laica part number ANE033):

Before each use, thoroughly clean and disinfect the atomisation chamber, lid, connection, mouthpiece and masks according to the paragraph "Cleaning and Maintenance".

Connect the transformer to the unit (mains outlet) and then the power outlet, making sure the voltage is

correct.

Unwrap the entire length of the cord, making sure no once can trip over the cord.

Attention: Only use the transformer provided by Laica, other transformers could damage the unit.

Make sure the power outlet does not come in contact with liquids.

If an extension cord or adapter is needed, make sure they are compliant with current safety regulations. The electrical capacity and maximum capacity specified on the adapter must not be exceeded. Proceed as described in points 3-9 in the paragraph "Using with batteries".

At the end of the treatment, turn the device off before unplugging the transformer from the mains. The transformer cannot be used to recharge batteries.

MICROBIAL CONTAMINATION

For illnesses where there is a risk of infection and microbial contamination, we recommend personal use of the accessories (atomisation chamber, mouthpiece, mask connection and masks). Always consult the physician.

CLEANING

- 1) **Attention! Hot surfaces! Before cleaning the unit, allow it to cool as the transducer and residual inhalation preparations become hot during use.** Always clean the unit after each use so that the residual medicine does not solidify and ruin the mechanism.
 - 2) Before cleaning operations remove the batteries or, if using with mains power, unplug.
 - 3) After each use, lift the lid on the atomisation chamber to empty any residues from the chamber itself. Clean the atomisation chamber and the atomisation dispenser as follows:
 - Pour tap water into the chamber, turn on the unit and atomise the water.
 - When the water is completely atomised, remove the atomisation chamber from the unit by pressing the attachment located on the bottom of the chamber itself (see fig.5).
 - Pour 10 ml of ethyl alcohol in the atomisation chamber, allow it to set for 10 minutes, then gently shake the atomisation chamber. Remove the ethyl alcohol.
 - Set the atomisation chamber on a flat surface, as indicated in figure 6, and pour 1-2 ml of ethyl alcohol in the atomisation dispenser.
 - Allow the alcohol to set for 10 minutes and then remove.
 - Place the atomisation chamber under running water and then let it air dry completely.
- Attention! Never use a cloth or any other object to clean the atomisation chamber.**
- Clean the electrical contacts on the base of the aerosol (upper side) and the atomisation chamber (lower side) with a cotton swab soaked in ethyl alcohol.
- Before hooking the atomisation chamber on the main unit, make sure it is perfectly dry.
- To hook it back on the appliance, position it on the main unit and lightly press down until you hear a click.
- 4) Clean the outside of the unit with a soft cloth dampened with water and neutral soap (do not clean the main unit by placing it under running water and make sure no liquids penetrate the appliance).
 - 5) We recommend disinfecting the accessories after each use to limit the risk of infections. In this case, rinse the mouthpiece, connection and masks under running water, then boil covered in water for about 10 minutes.

Once disinfection is complete, dry the previously boiled accessories with gauze, being careful not to burn yourself.

Do not dry the accessories with cloth in order to avoid damaging them.

- 6) **Disinfect all of the components before using the appliance for the first time.**
- 7) The appliance, atomisation chamber and all accessories should be perfectly dry before use.
- 8) Store the appliance and all accessories in the provided case, in a cool dry location.

TROUBLESHOOTING

- The supply of steam may be interrupted in the following cases:
 - If during the treatment, the device has been tilted more than 45°C. Keep the device in the correct position during inhalation. Re-read the paragraph INSTRUCTIONS FOR USE.
 - The drug present in the atomisation chamber has almost run out. The appliance has an automatic stop device that disables the unit when the medicine is almost finished.
 - The batteries are low. When the orange warning light switches on, do not switch on the device, as this can cause damage. Replace batteries or connect the appliance to the mains (if you have purchased the transformer).
 - There are bubbles in the liquid to be inhaled. Eliminate any bubbles in the atomisation chamber by gently shaking the aerosol back and forth and then press the "O/I" button.
- The appliance does not operate correctly after a long period of non-use.
 - Make sure there are no drug residues in the atomisation chamber. Clean and dry the atomisation chamber as described in the paragraph on cleaning.
 - Make sure there are no drug residues in the atomisation dispenser and the mask connections and mouthpiece. To check, fill the atomisation chamber with 6 ml of water and turn on the appliance. If after 30 minutes there is still water in the atomisation chamber, then the openings are blocked.

In this case, proceed as follows:

- 1) Fill the atomisation chamber with 6 ml of water and add 2 drops of vinegar.
 - 2) Turn the appliance on and wait until atomisation is complete.
 - 3) Clean the appliance as described in the paragraph on cleaning.
- If the appliance still does not atomise correctly after performing this operation, you can proceed as follows (only if the previous remedy did not work):
- 1) Tilt the aerosol 90°C and put a dry cloth under the device as it could drip during this operation.
 - 2) Hold down the "O/I" button until the light flashes purple 1 time.
 - 3) Pour 2 ml of water in the dispenser so atomisation occurs inside the atomisation chamber (see fig.7).
 - 4) Turn off the unit by pressing the "O/I" button as soon as there is no longer any water in the dispenser because in inverse mode the appliance does not turn off automatically.

ELECTROMAGNETIC COMPATIBILITY

This device complies with current regulations relating to electromagnetic compatibility (EMC) and has been designed to be used in the home.

Its emissions are extremely small and are unlikely to generate interference with other equipment. In the event

that it is to be used in close proximity to other equipment, it is recommended to follow the instructions given in the tables at the end of the instructions for use.

Certain mobile communications devices and radio systems may have an effect on the operation of this device. In cases where there are anomalies during operation, it is recommended to distance the device from any other equipment that could cause electromagnetic interference and check for the restoration of performance. In all cases, in the event of doubt, contact Customer Support or see the tables relating to electromagnetic emissions at the end of these instructions.

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m2 without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This appliance is guaranteed for 2 years from the time of delivery of the goods, or another longer term envisaged by the national legislation of the consumer's residence. This provision complies with Italian and European legislation. The Laica products are designed for home use and must not be used in public venues. The warranty only covers manufacturing defects and does not apply if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. Use only the accessories supplied; the use of different accessories may result in invalidity of the warranty. Do not open the unit for any reason; in the case of opening or tampering, the warranty is definitively voided. This warranty does not apply to parts subject to wear or to the batteries when supplied. After 2 years from delivery, or another longer term envisaged by the national legislation of the consumer's residence, the warranty expires; in this case, the technical assistance interventions will be carried out against a fee. Information on technical assistance, whether under warranty or for a fee, can be requested by contacting info@laica.com.

No payment will be due for repairs or replacements of products that fall within the terms of the warranty. In the event of faults, contact the retailer. Do NOT send the appliance directly to LAICA. All the operations under warranty (including those of replacement of the product or part thereof) will not prolong the duration of the original period of warranty of the product replaced. The manufacturer declines any liability for any damage that may, directly or indirectly, be caused to persons, property or animals as a result of the non-observance of all the requirements established in the relevant instructions manual and concerning, especially, warnings relating to installation, use and maintenance of the appliance. Laica, in its constant commitment to improving its products, is entitled to changing without any notice, in whole or in part, its products in relation to production requirements, without this entailing any liability for Laica towards its dealers. For further information: www.laica.it.

Electromagnetic compatibility		
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The DEVICE uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. It is possible to use the device in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings user for domestic purposes.
RF emissions CISPR 14	Class B Conforms	
Harmonic emissions IEC 61000-3-2	Class C Conforms	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Immunity aspects			
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.			
Immunity test	Test level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) EN 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Burst/Fast Transient EN 61000-4-4	±2kV power supply lines	±2kV power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge EN 61000-4-5	±1kV differentail mode ±2kV common mode	±1kV differentail mode ±2kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines EN 61000-4-11	< 5% UT (>95% dip in UT) for 0,5 cycles 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (>95% dip in UT) for 5 seconds	< 5% UT (>95% dip in UT) for 0,5 cycles 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles < 5% UT (>95% dip in UT) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions. It is recommended that the DEVICE be powered from an uninterruptible power supply or a battery.
Power frequency magnetic field EN 61000-4-8	3 A/m	NA	NA
Note: UT is the a.c. mains voltage prior to application of the test level.			

Immunity aspects at r.f.			
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.			
Immunity test	Test Level EN 60601-1-2	Compliance level	Electromagnetic environment – guidance
RF conducted EN 61000-4-6	3 Veff from 150kHz to 80MHz	3 V/m	Portable and mobile RF communications equipment shouldbe used no closer to any part of the device, including cables, than the recommended separation distance calculated from that equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).
RF radiated EN 61000-4-3	3 V/m from 80MHz to 2,5GHz	3 V/m	
Field strengths from fixed RF transmitters, as determined by an electromagnetic site survery, c should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 			

Recommended separation distances between portable and mobile RF communications equipment and the DEVICE			
The DEVICE is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the DEVICE can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the DEVICE as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the DEVICE is used exceeds the applicable RF compliance level above, the DEVICE should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the DEVICE.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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