## IT Italiano

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### ELECTRONIC PERSONAL SCALE

Istruccions and warranty

Your new bathroom scale is extremely reliable and precise, it does not require any sort of maintenance. To improve the reading, weigh yourself twice, and if the weights are different, your weight is the middle of the two readings. Always weigh yourself on the same scale, at the same time of day, preferably undressed and before breakfast. Warning! The surface of the scale can be slippery, if wet.

**IT IS IMPORTANT TO READ THE INSTRUCTIONS AND WARNINGS IN THIS HANDBOOK BEFORE USE AND KEEP IT IN A SAFE PLACE.**

#### THE BODY WEIGHT

Checking the weight means performing an **energetic balance**: such balance results from the ratio between the quantity of ingested nutrients, i.e. the ingested calories, and the actually consumed calories. The daily necessary quantity of energy depends on the metabolism, on the life-style and on the type of activities performed by each single person. Controlling one's own weight is essential in order to protect health effectively: eating well means being healthy. It has been proved that most of diseases, many of which extremely dangerous, are associated to unbalanced eating habits not only in terms of quality but also in terms of quantity. Among the most known, the following ones should be mentioned:

- ischemic heart diseases;

- cerebrovascular diseases;

- diabetes mellitus;

- tumores tumors;

- etc.

All these diseases are present above all among obese persons; therefore, controlling the weight together with controlling the quality of the nutrients becomes one of the essential factors to prevent diseases. Healthy eating means selecting the foodstuffs that supply the body with all the necessary nutriment to grow and keep the physiological functions and the biochemical processes balanced.

#### Hints for a healthy feeding and to control weight:

- To follow a balanced diet it is necessary not to exceed in the consumption of the same types of food neglecting the essential elements contained in other types of food, as well as not eating more than needed. A healthy diet includes suitable quantities of proteins, carbohydrates, fibers, vitamins and minerals, as well as, obviously, water.
- Having a complete meal only once a day is a bad habit. It is better to have many small meals instead of just an abundant one. Three meals a day and a sneak in the middle of the morning or of the afternoon represent the basis for a correct feeding.
- It is suggested to sit down and relax when eating. Select a comfortable environment can be relevant.
- When you are focused on doing other activities, i.e. when you are watching TV or working, in order to avoid eating too much without even noticing it. To support the digestion process it is recommended to eat slowly and chew well, enjoying the color, taste and consistency of the food.
- Combining a correct feeding to physical activity is an essential condition to allow increasing the metabolism, consuming more calories and remaining in good shape.

Getting used to follow a balanced diet is surely of help in increasing the quality of life, promoting health and preventing diseases, as well as in increasing vitality and remaining in good shape!

#### Caution!!

- Never follow diets autonomously, always refer to a doctor or dietician.**
- Self-measurement means control, not diagnosis or treatment. Unusual values have always to be discussed with one's own physician.**

**Under no circumstances must you change the dose of medicines prescribed by your Doctor.**

#### ISTRUCTIONS FOR USE

- Select the weighing unit of measure (kg, lb or st) with the key inside the battery compartment located on the bottom of the scale. It is possible to select the unit of measure if the scale is turned off as well as if it is turned on.
- Place the scale on a flat, hard surface. If the floor is loose or uneven (e.g. mats, carpets, linoleum) the exactness of your weight is not guaranteed.**
- Step onto the scale with your feet parallel and your weight equally distributed (see illustrations). Warning: the scale switches on once step on. Stand still while the scale measures your weight. Do not lean on anything while being weighed as this could alter the reading.**
- Step off the scale; the display screen will continue to show your weight for a few seconds then the scale will turn itself off automatically.

Each time the scale is moved and then returned to its resting surface, the first weight must not be considered due to reorganization of the electronic board with the loading cells. Consider the second and successive weights as in all certain cases the first one could be altered for the reason mentioned above.

#### WARNING:

- When the display screen shows "LO" or the symbol of the battery, it's necessary to replace the battery.
- In case of excess weight, the display screen shows "Err", "O-Ld", "-----" or "FFFF", remove the weight from the scale to avoid any damage of the mechanism.

#### BATTERIES

This scale operates with one 3V replaceable lithium battery, type CR2032. Remove the protective film from the batteries before use. Remove the battery if the scale is not to be used for a prolonged period of time. Remove the run down battery by lifting the cover of the battery compartment located on the bottom of the device. Extract the battery and dispose it as special waste at the specific collection points forecast for recycling. For more information about the disposal of run down batteries, please contact either the store where the device containing the batteries was purchased or the retailer who provided the warranty. Do not open the battery compartment. Do not attempt to repair the battery. **Caution: For greater safety, children younger than 12 years should not remove the batteries.**

#### MAINTENANCE

- Handle your weight with care as it is a precision instrument. Therefore, avoid to expose it to extreme temperatures, damp, knocks, dust and vibrations. Do not use the scale on wet or slippery floors.
- Keep the scale away from sources of heat.
- Do not jump on the scale.**
- Clean the platform with a damp cloth.**
- Clean the bottom of the scale with just a soft dry cloth.**
- To clean the scale never use chemical or abrasive products.**
- Take care that no liquids penetrate into the scale, as it is not waterproof. Do not use the scale in very damp surroundings, humidity above 85% or contact with water and other liquid parts could cause corrosion and jeopardize the precision of the scale.

#### TECHNICAL FEATURES

One 3V CR2032 lithium battery

LCD display

Operating environmental requirements: +10°C +40°C; RH ≤85%

Storage environmental requirements: +10°C +50°C; RH ≤85%

- Important! Read the instructions carefully

## EN English

### ELECTRONIC PERSONAL SCALE

Istruccions and warranty

This product is meant for domestic use. The conformity, certified by the CE-marking on the device, refers to the Directive 30/2014/Eu on electromagnetic compatibility. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

#### DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of the life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m<sup>2</sup> without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices carried out following a European environmental policy aimed at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

#### WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means of **the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document**. This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA. **All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.** The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and

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